



CLINICAL GUIDE TO SET UP

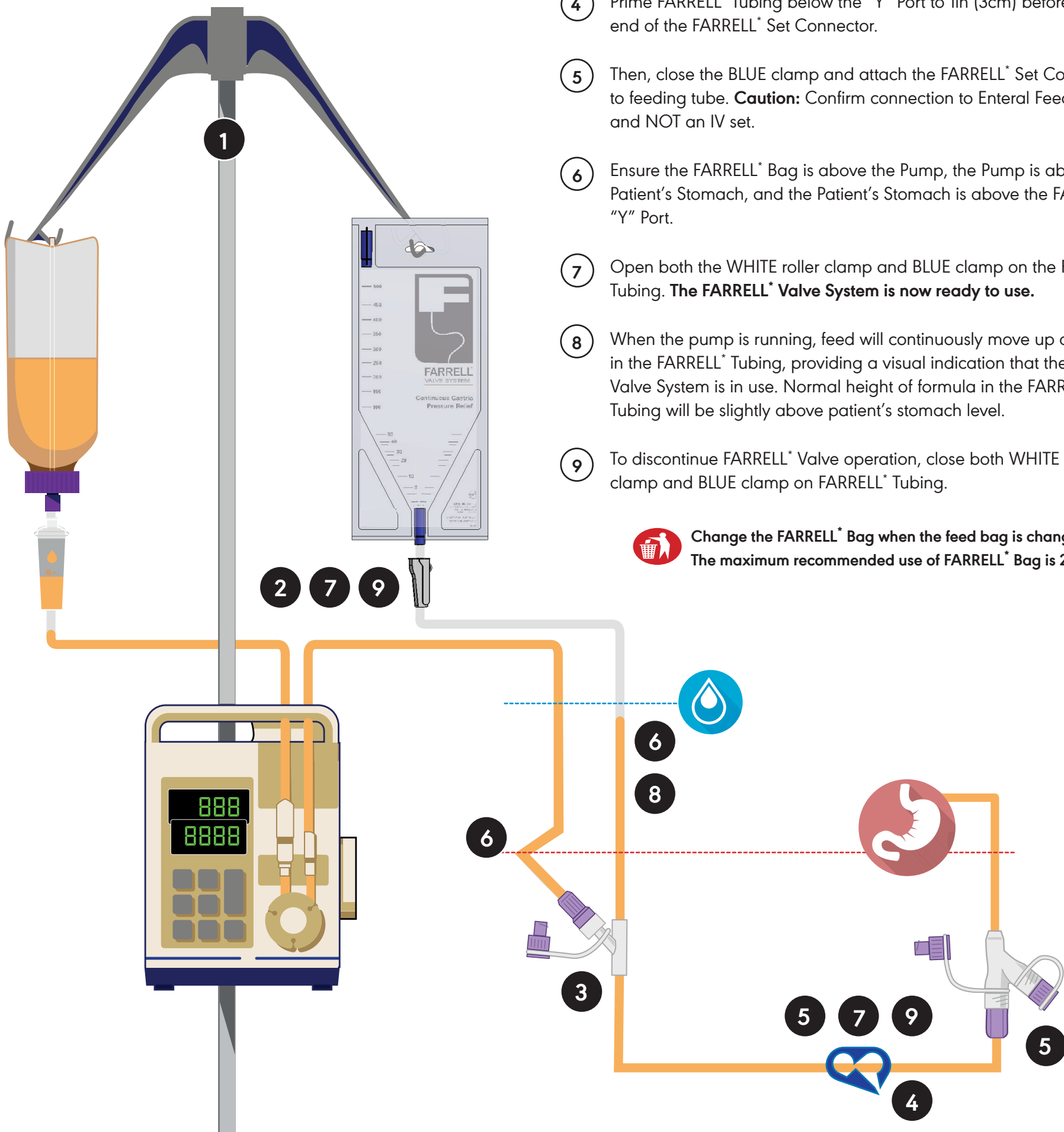

- 1 Hang FARRELL* Bag at the same height as the feeding bag/container.
NOTE: Prime pump according to pump instructions.
- 2 Close the WHITE clamp above FARRELL* "Y" Port.
- 3 Attach the administration set to the FARRELL* "Y" Port.
NOTE: For syringe feed, attach syringe to FARRELL* "Y" Port.
- 4 Prime FARRELL* Tubing below the "Y" Port to 1in (3cm) before the distal end of the FARRELL* Set Connector.
- 5 Then, close the BLUE clamp and attach the FARRELL* Set Connector to feeding tube. **Caution:** Confirm connection to Enteral Feeding Tube and NOT an IV set.
- 6 Ensure the FARRELL* Bag is above the Pump, the Pump is above the Patient's Stomach, and the Patient's Stomach is above the FARRELL* "Y" Port.
- 7 Open both the WHITE roller clamp and BLUE clamp on the FARRELL* Tubing. **The FARRELL* Valve System is now ready to use.**
- 8 When the pump is running, feed will continuously move up and down in the FARRELL* Tubing, providing a visual indication that the FARRELL* Valve System is in use. Normal height of formula in the FARRELL* Tubing will be slightly above patient's stomach level.
- 9 To discontinue FARRELL* Valve operation, close both WHITE roller clamp and BLUE clamp on FARRELL* Tubing.

 **Change the FARRELL* Bag when the feed bag is changed.**
The maximum recommended use of FARRELL* Bag is 24 hours.

TOP TIPS

- In order for the FARRELL* Valve System to work properly, the FARRELL* "Y" Port must be AT or BELOW the patient's stomach.
- If a patient has a distended stomach, manually decompress the patient's stomach with a syringe prior to the initial use of the FARRELL* Valve System.
- Patients on very low volume feeds (Ex. neonates) may require substantially longer time (60-90 minutes) to establish flow before opening the WHITE clamp above the "Y" Port.

- When administering medication, use the access port on the feeding tube if possible. Close the BLUE clamp before opening the feeding tube access port. Administer medication and wait 5-10 minutes before closing the feeding tube access port and re-opening the BLUE clamp.
- If a feeding tube occlusion is suspected, then close the BLUE clamp and disconnect the FARRELL* Valve System. Check for occlusions in the feeding tube. Once the occlusion is resolved reconnect the FARRELL* Valve System.

There are inherent risks in all medical devices. Please refer to the product labeling for Indications, Cautions, Warnings and Contraindications.